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PORTABLE ELECTRIC BREAST PUMP

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TRAGBARE ELEKTRISCHE MILCHPUMPE

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TIRA-LEITE ELÉTRICO PORTÁTIL

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EN INSTRUCTIONS FOR USE

ELECTRIC BREAST PUMP

Please read these instructions carefully before use and keep them for future reference.

Intended use and indications for the electric breast pump

SAFETY WARNINGS

WARNING!

- This product is intended for domestic use only by an adult, and must be used in accordance with the instructions for use provided. Any other use is to be deemed improper and therefore dangerous. Artsana declines all responsibility for any damage caused by inappropriate, incorrect or unreasonable use of the appliance.
- The appliance is for personal use and must not be sold by the purchaser or lent to other users.
- This appliance and its components must not be exposed to direct sunlight, rain and other elements. The mains adaptor is for indoor use only and should therefore not be exposed to sunlight, rain or other elements.
- If the appliance is used outdoors, it must be powered only by internal batteries and must not in any case be exposed to sunlight, rain or other elements: failure to comply with this warning may expose the user to risk of electric shock.
- This appliance is not a toy. This appliance should not be used by children or persons with reduced mental faculties, or who do not have the required experience or knowledge. The product can be used by persons with reduced physical or sensory capacity as long as under supervision or after they have received instructions for the safe use of the appliance, and have understood the hazards involved.
- The appliance must be powered at a safety low voltage.
- Keep the appliance, its components and the USB cable out of the reach of children. The USB cable and the tube connecting the motor unit to the extraction kit, if left within children's reach, may cause strangulation. The appliance also contains small parts which, if ingested by a child, may cause suffocation.
- Please ensure that children do not play with the appliance, its components or the USB cable.
- Always place the appliance only on a flat, stable surface, away from cribs and cots.
- Do not perform any cleaning or sanitising operations on the breast pump while it is operating or recharging. Cleaning and sanitising operations must be carried out in accordance with these instructions for use.

- In order to avoid the proliferation of bacteria, it is advisable to feed the baby immediately after the milk has been collected, or to immediately store the collected milk in the most appropriate way.
- Clean and sanitise the silicone cushion (3), the pump body (4) and the valve (5) before each single use. Also clean and sanitise the feeding bottle (6) and its accessories (7)-(8), according to the instructions for the bottle.
- Before each use, check that the membrane is not damaged: it must never be torn or cut. If it is torn or cut, do not use the appliance.
- The cleaning and sanitising to be performed by the user must not be carried out by children and/or people with reduced mental capabilities or lacking experience or the required knowledge.
- Do not wet the appliance or the USB cable or put them into water or other liquids and never place them under running water. If any liquid has entered the appliance or the USB cable, contact our qualified technical personnel before further use.
- If the appliance or the USB cable show any sign of damage, they must not be used. If the appliance or the USB cable show any sign of damage, contact our qualified technical personnel before further use.
- Non-replaceable battery.
- Do not attempt to open or repair the appliance if faulty or damaged. In the event of faults or damage to the appliance in general, to the motor unit or to the USB cable contact our qualified technical personnel before further use. Any repair/service carried out directly by the user may lead to severe risks to her or to any third party's safety as well as irreparably damaging the appliance and entails immediate termination of the warranty.
- If the packaging appears to be damaged, check the state of all the components and parts of the appliance. If the USB cable and the motor unit proved to be undamaged, wash all the components of the kit following the instructions given in the paragraph "CLEANING" as a precaution.
- Before use, remove and dispose of all plastic bags and packaging materials and keep them out of reach of children; they may be a source of danger.
- After removing any packaging materials, ensure that the appliance has not been damaged. If in any doubt, do not use the appliance and contact only technically qualified personnel, at the point of purchase.
- Make sure that the appliance is placed on a level, stable surface when used in order to avoid spillage and prevent it from falling.
- Do not place the appliance on delicate surfaces or surfaces that may be sensitive to humidity and heat.
- Do not use the appliance while taking a bath or shower, or when in a damp place, or close to bath-tubs, sinks, washbasins, or in any other situation in which there are liquids that may come into contact with the appliance.
- Make sure that, when in use, the kit is not too from the motor unit so that the latter does not fall.

- When using any electrical appliance, it is necessary to follow a few basic safety rules:
 - Never touch the appliance with wet or damp hands.
 - Keep away from heat sources.
 - If the appliance and/or the USB cable are faulty and/or not operating correctly, immediately switch off the appliance, do not attempt to tamper with or use the appliance but contact qualified personnel.
- The appliance must be cleaned and dried thoroughly at the end of every operating cycle.
- If the appliance is not going to be used for a prolonged period, it should be thoroughly cleaned and the batteries (if installed) removed before storing it in a clean and dry place away from light and heat sources and according to the storage and transport instructions.
- If you decide not to use the appliance any more, do not dispose of it as normal urban waste but as waste electrical and electronic equipment (WEEE) as required by EU Directive 2012/19/EU or by the laws in force in the countries where it is sold.
- To avoid any risk of fire or electric shock, ensure that the USB cable does not pass under any carpets or close to radiators, stoves and heaters.
- Operate the appliance at a temperature range between +5°C and +40°C. Use outside these temperature limits, and in particular above the upper limit, may cause the appliance to overheat with consequent damage to it and/or risk of burns or access to live parts of it by the user.
- The device was designed to only work with parts supplied in the original pack and with 150 ml capacity feeding bottles. It may not perform as declared if used with other parts or feeding bottles of greater capacity.
- The appliance must not be used continuously for more than 30 minutes. When you have finished using it, the appliance must be switched off for at least 30 minutes.
- Product features may be subjected to change without notice.
- If you have any doubts on interpretation of the contents of this instruction booklet, contact your dealer.

RECOMMENDATIONS ON THE LITHIUM BATTERY AND PRODUCT CHARGING

- It is recommended that you charge the product overnight when using it the first time.
- Follow the instructions in the manual to properly charge the lithium battery.
- Keep the product out of reach of children whilst charging.
- The lithium battery is not user-replaceable and in any case the use of a lithium battery other than the one supplied is not permitted.
- Recharging must be carried out only by means of a USB cable directly connected to a PC.
- Never use the product for any use other than the one for which it was designed.
- Do not charge the lithium battery when not in use.
- ARTSANA declines all responsibility for any damage caused by inappropriate, incorrect or unreasonable use of the appliance.
- Dispose of properly as indicated in the manual.

INDEX OF SYMBOLS

	Product catalogue reference code
	CAUTION!
	For indoor use only
	ON/OFF (press-press)
	Alternating current
	Direct Current
	Complies with European Directive 2012/19/EU

THE BREAST PUMP AND BREASTFEEDING

Chicco's electric breast pump helps extend breastfeeding, in the event of excess milk or when you cannot always be with your baby. In such cases you can extract milk, store it, and give it to baby when required.

For maximum flexibility and convenience, the electric Breast Pump works in both battery mode (lithium battery supplied) and battery recharge mode, via USB cable directly connected to the PC.

LIST OF COMPONENTS

- 1- Membrane Cod 20091990000
- 2- Membrane cover Cod 20423080000
- 3- Soft silicone cover Cod 21174540000
- 4- Pump body Cod 20423070000
- 5- Valve Cod 20067541600
- 6- NaturalFeeling feeding bottle Cod 21600730000
- 7- 0m teat with cap and ring Cod 21174393012 Cod 20000001127 Cod 21174400000
- 8- Airtight cap Cod 20174300003
- 9- USB cable plug
- 10- USB cable connector
- 11- Silicone tube inlet
- 12- USB cable input
- 13- Motor unit
- 14- LED indicator
- 15- On/Off key
- 16- Stimulation/extraction button
- 17- Button for increasing level of extraction/stimulation
- 18- Button for reducing level of extraction/stimulation
- 19- Silicone tube

SET-UP

- Before using the Breast pump for the first time, please clean all the components* thoroughly as shown in the "**CLEANING**" paragraph.
 - Then assemble the breast pump as follows:
 - Push the valve (5) down into the circular duct in the base of the pump body (4). (Fig. 1)
- Note: fit the valve preferably with the tab positioned inside the pump body.
- Push the outer edge to prevent damaging the valve and

make sure that both closing edges of the assembled valve are joined together. If not, reposition the valve, making sure that the two edges are correctly aligned.

- For greater comfort, you can fit the silicone cover (3) onto the pump body (4). Insert the silicone cover well into the cone of the pump body, and seal it around the outer rim of the pump body (Fig.2). Press the pads of the cover to expel any air trapped inside.
 - Fit the membrane cover (2) onto the silicone membrane (1) and insert the membrane and cover (pre-assembled) onto the pump body (4) (Fig.3)
 - Screw the NaturalFeeling feeding bottle (6) onto the threaded base of the pump body (4).(Fig.4)
 - Use the practical base to lay the assembled product on a flat, stable surface.
 - To use or charge the breast pump with its USB cable, attach the USB cable plug (9) to the USB cable input on the motor body (12) and then connect the USB connector of the USB cable (10) to the PC as indicated in this manual.
 - Connect the silicone tube (19) to the membrane cover (2) (Fig.6).
 - Insert the end of the silicone tube (19) into the appropriate inlet on the motor body (11) (Fig.7).
- *Excluding the motor body (12)(Fig. 5), USB cable (9,10) and silicone tube (19).

USING THE BREAST PUMP

- Before each use, sanitise all the components* with a cold sanitizing method using sterilising liquids (following the manufacturer's instructions), or a hot one (steam) using specific devices, or by boiling them (5 minutes), and letting all the hot sanitized components fully cool down before use.
- Wash your hands and assemble the breast pump as shown in the **PREPARATION** paragraph.
- Cleanse your breast appropriately using water or one of the handy Chicco breast cleansing wipes
- Sit down comfortably and relax.

Note: while using the breast pump, to ensure correct operation of the appliance, take care not to cover the hole on the left-hand side of the pump body (4) with your hand.

Note: when switched on, the breast pump always starts

in stimulation mode to facilitate milk extraction, using the factory settings (level 1). The led indicator light (14) is blue and steady.

Note: To avoid all danger deriving from spilt liquid (milk), we recommend that you never fill the feeding bottle beyond the highest level indicated on its graduated scale.

- Fit the cup over your breast ensuring that the nipple is centred properly. You can use the breast pump both with and without the soft silicone cover (3); We recommend trying both configurations before choosing the best and most comfortable one for you.
- In battery mode, switch on the breast pump by pressing the on/off button (15) on the motor body (13), the LED indicator (14) will switch on with a steady blue light.
- After approximately 3 minutes of stimulation, the appliance automatically switches to the extraction mode (factory setting) and the LED indicator switches on with a white light.
- You can stop stimulation at any moment and switch to the expression stage and vice versa by pressing the stimulation/extraction button (16).
- If necessary, repeat stimulation by pressing the stimulation/extraction button (16) again. Press the stimulation/extraction button (16) to return to extraction.
- You can use the "+" (17) and "-" (18) buttons on the keypad to adjust stimulation/extraction based on the level of pull perceived on your breast and/or based on the amount of milk expressed. Each time you change level, the LED indicator (14) will blink once with a blue or white light depending on the chosen mode. We recommend adjusting the extraction/stimulation rate gradually until you reach the level best suited to you.
- If the level of extraction/stimulation chosen causes discomfort or pain, reduce it to a suitable level by pressing the "-" button (18).
- You can alternate milk extraction from both breasts.
- Press the on/off button (15) to switch the breast pump off.

WARNING: After 30 minutes of use, the appliance switches off automatically for safety reasons.

In case of low battery the LED indicator light (14) will start blinking white during extraction mode, and blue during stimulation mode: in this case it is recommended to recharge the battery and, if necessary, to use the product in battery charging mode. Otherwise, the appliance will switch off automatically after a few minutes.

CHARGING THE BATTERY

To charge, connect the motor body (13) to a PC using the USB cable. To reduce charging time, it is recommended to charge with the appliance switched off.

During charging with the USB cable inserted and the product switched off, the battery charging status will be indicated by the LED indicator (14) lit with a steady white light, the fully charged status by the LED indicator (14) lit with a steady blue light.

When using the appliance while charging the battery, it is not possible to indicate the appliance's charge status.

- **WARNING:** immediately after each single use, carefully clean all the components* as shown in the "CLEANING" paragraph.
- To feed the milk immediately to your baby, unscrew the feeding bottle from the pump body (4) to remove it.

Screw the ring with the teat (7) (included in the pack) onto the feeding bottle. This way, you will obtain a handy, ready-to-use baby feeding bottle.

- If, on the contrary, you close the bottle with the airtight cap (8) (included in the package) you can store the expressed milk in the fridge for up to 48 hours at 4°C or for up to 3 months at -18°C in the freezer.

- **WARNING:** When the breast pump is not used, always disconnect the USB cable from the motor body and PC.

*Excluding the motor body (13), USB cable (9,10) and silicone tube (19).

CLEANING

- Before carrying out any cleaning operation, disconnect, if connected, the USB cable from the PC and the USB cable plug (9) from the USB cable input (12) on the motor body.
- Separate all the components from the breast pump.

Note: remove the valve (5), located on the circular duct inside the base of the pump body (4) by gently pulling the tab on the side of the valve.

- **WARNING:** Make sure that the valve is always removed in a gentle way; do not use any tool for cleaning it because it may be damaged, making the breast pump impossible to use.

Note: to remove the silicone tube, first detach the membrane (1) and the membrane cover from the pump body, to avoid any spillage. Then, holding the end inserted into the membrane cover, gently pull it off.

- Carefully wash all the components* using hot water and washing up liquid, thoroughly rinse and dry carefully using a soft and clean cloth.
- Do not use abrasive products to clean the breast pump and its components.
- Clean the motor body with a soft, clean and dry cloth. Warning! Do not wet the appliance or put it into water or other liquids.
- The tube may be rinsed and suitably dried.

- All the components, with the exception of the motor body (13), the USB cable, the valve (5), the membrane (1), the silicone cover (3) and the tube (19) may also be washed in a dishwasher using a delicate programme. This washing method, which is more aggressive than hand washing, may discolour or dull and degrade the material over time.
- Also clean and sanitise the feeding bottle (6) and its accessories (7)-(8), according to the instructions included with the bottle.
- After washing, always store the product in a clean, dry place.

*Excluding the motor body (13), USB cable (9,10) and silicone tube (19).

ACCESSORIES:

- NaturalFeeling feeding bottle 150 ml [REF](#) 00080711000xxx / [REF](#) 00080811000xxx.

- NaturalFeeling milk container [REF](#) 00002257200000

- Where XXX=variable identification code identifying the country of sale.

- No accessories other than those recommended must be used.

Trouble	Cause	Solution
The breast pump does not pump or pumps very little.	Valve not assembled or not properly assembled.	Assemble the valve correctly.
	Cup and motor body not assembly correctly.	Check that you have performed the procedure described in the "PREPARATION" paragraph correctly. Make sure that the silicone tube is firmly connected to both the motor unit and the membrane cover. Check that the tube is not damaged. Do not twist the tube during milk expression.
	The cup does not fit well on the breast.	Fit the pump body over your breast, taking care that your nipple is properly centred. Try with/without the soft silicone cover.
	Suction level not correctly adjusted.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "+" or "-" buttons.
	Low or insufficiently charged battery	Charge the battery
Milk expression is painful.	Suction level too high.	Change the suction level until you achieve more comfortable stimulation and suction by pressing the "-" button.
The breast pump does not switch on.	USB cable/appliance/PC USB port faulty	Check that mains power is present in the PC
	Incorrect connections.	Check all the connections.
	Low battery.	Charge the battery

TECHNICAL SPECIFICATIONS

- REF 00009199500000 PORTABLE ELECTRIC BREAST PUMP Mod. N0002
- Maximum vacuum level in extraction mode: $-0.35 \pm 0.03 \text{ bar}^{**}$ (** measured using a sample cylinder).
- Input voltage: 5V  2A
- 3.7V 1100mAh lithium battery 
- Appliance service life: 500 hours
- Minimum operating time with battery: 90 minutes
- Type of appliance: intermittent operation; 30 minutes ON - 30 minutes OFF
- Made in China

Operating conditions:

- Operating temperature: $+5^{\circ}\text{C} / +40^{\circ}\text{C}$

DECLARATION OF EC COMPLIANCE

This Chicco brand Electric Breast Pump appliance REF 00009199500000 Mod. N0002 complies with all the essential requirements and other provisions indicated in the EU Directives 2014/30/EC (electromagnetic compatibility), 2014/35/EC (Low Voltage), and 2011/65/EC (RoHS). A copy of the EC declaration can be requested from the manufacturer: Artsana S.p.A via Saldarini Catelli, 1 - 22070 Grandate (Como) Italy.

THIS PRODUCT COMPLIES WITH THE DIRECTIVE 2012/19/EU.

The crossed bin symbol on the appliance indicates that the product, at the end of its life, must be disposed of separately from domestic waste, either by taking it to a separate waste disposal site for electric and electronic appliances or by returning it to your dealer when you buy another similar appliance. The user is responsible for taking the appliance to a special waste disposal site at the end of its life. If the unwanted appliance is collected correctly as separate waste, it can be recycled, treated and disposed of ecologically; this avoids a negative impact on both

the environment and health, and contributes towards the recycling of the product's materials. . For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

COMPLIANCE WITH EU DIRECTIVE 2006/66 CE

The crossed bin symbol on the batteries or product pack indicates that, at the end of their life, they must not be disposed of as urban refuse. They must be disposed of separately from domestic waste, either by taking them to a separate waste disposal site for batteries or by returning them to your dealer when you buy similar rechargeable or non-rechargeable batteries. The chemical symbols Hg, Cd, Pb, printed under the crossed bin symbol, indicate the type of substance contained in the batteries: Hg=Mercury, Cd=Cadmium, Pb=Lead. The user is responsible for taking the batteries to special waste disposal sites at the end of their life to allow for their processing and recycling. If dead batteries are collected correctly as separate waste they can be recycled, processed and disposed of ecologically; this avoids a negative impact on both the environment and human health, and contributes towards the recycling of the substances contained in the batteries. Improper disposal of the product by the user may harm the environment and human health. For further information regarding the waste disposal services available, contact your local waste disposal agency or the shop where you bought the appliance.

WARRANTY

The product is guaranteed against any conformity defect in normal conditions of use as provided for by the instructions for use. The warranty shall not therefore apply in the case of damage caused by improper use, wear or accidental events. For the duration of warranty on conformity defects please refer to the specific provisions of national laws

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Artsana S.p.A – via Saldarini Catelli, 1 - 22070 Grandate (Como) – Italy